

- Biocompatible-made of all natural chitosan polymers
- Easily conforms to nasal anatomy
- Offers easier placement with a superior field of view
- Minimizes bleeding and edema after surgery
- No special storage or refrigeration required




PosiSep®X², is indicated for use as a hemostat dressing/intranasal splint intended to minimize bleeding and edema and to prevent adhesions between the septum and the nasal cavity. **PosiSep®X²** is constructed from proprietary chitosan polymers in a patient-comfortable sponge which is placed in the nasal cavity after surgery or trauma. After hydration, **PosiSep®X²** will rapidly expand to fill the space between mucosal tissues and prevent post-op adhesions. Subsequent irrigation will cause the product to fragment, thus optimizing patient tolerance and comfort. Typical clearance time is about 5-10 days.

- ✓ Fragments by way of gentle irrigation
- ✓ Easy to apply-no mixing or additional prep time necessary
- ✓ Widely studied and published anti-adhesion properties of our proprietary NOCC technology

What is NOCC?

Chitosan is widely used in wound treatment technologies due to its biomedical properties. It is made from Chitin an amino-polysaccharide removed from the powdered shells of crustaceans. NOCC is our proprietary medical grade NO-Carboxymethyl Chitosan derived from Chitosan. NOCC has unique properties which make it advantageous for use in medical applications.

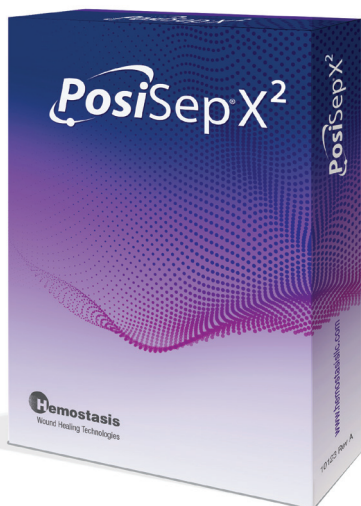

When PosiSep®X² is hydrated, it expands to fill the space preventing post-op adhesions.

ORDERING INFORMATION

Part Number:	Description:
9210614	PosiSep®X ² Hemostat Dressing/Intranasal Splint 0.7" x 1.7" 5pk (US)
9210615	PosiSep®X ² Hemostat Dressing/Intranasal Splint 0.8" x 2.3" 5pk (US)

How Supplied:

PosiSep®X² is packaged with (5) sterile, single use units per carton. One (1) sterile unit contains one (1) Hemostat Dressing/Intranasal Splint. PosiSep®X² is terminally sterilized by gamma irradiation.

CAUTION: Federal U.S. Law restricts this device to sale by or on the order of a physician. PosiSep®X is contraindicated for patients who have known allergies to shellfish. For detailed information regarding indications for use, warnings and precautions, see instructions for use (IFU).

To place an order or for more information on PosiSep® or our other medical technologies, please contact a sales representative by calling our toll free number 866-612-2568